

PREFACE

Notice

Xuzhou Yongkang Electronic Science Technology Co., Ltd makes no warranty in any form, including but not limited to implied warranty of marketability and suitability provided it for a specific purpose. Xuzhou Yongkang Electronic Science Technology Co., Ltd assumes no responsibility for the error contained in this data, accidental or indirect damages due to the supply of this manual or caused by actual performance and utilization. This manual includes special data protected by copyright law. All rights reserved. Without prior written consent of Xuzhou Yongkang Electronic Science Technology Co., Ltd, any part of this manual shall not be photocopied, copied or translated into other language. The content contained in the manual can be changed without notice.

Responsibilities of manufacturer

Xuzhou Yongkang Electronic Science Technology Co., Ltd considers that it shall be responsible for the safety and reliability objectively existed in the instrument only under the following situations, namely assembly operation and repair are conducted by personnel recognized by the company and the instrument is used in accordance with the operation guide.

Warning:

The intended use of this instrument is clinical application, not treatment. If the result of fetal heart rate is unbelievable, please use other methods immediately, for instance, use stethoscope to conduct verification.

Explanation of annotations in this manual

Warning:

The information you should understand on how to avoid possible damages of patients and medical staff.

Caution:

The information you should understand on how to avoid possible damages of equipment.

Notice:

Important information you should understand.

1

CHAPTER 1 SAFETY GUIDE

1.1 Instructions for safety in use

To avoid possible damages, please abide by the following instructions for safety in use when operating this instrument.

Warning:

Do not use it at places with combustible gas, for instance, anesthetics.

Warning:

Please do not throw battery in fire so as to avoid explosion.

Warning:

Please do not touch signal input or output connector at the same time or patient.

Warning:

Battery shall be replaced in the environment away from patient (about 1.5 meters away from patient).

Warning:

Wastes from replacing scrapped components and aging components shall be disposed in accordance with national management provisions for medical wastes and relevant local laws and regulations for scrapped electronic components. Instead of random discarding, they shall be disposed uniformly so as to avoid pollution to environment.

Caution:

This instrument shall be repaired by authorized qualified engineer.

Caution:

This instrument is designed to the continuous operation type of common equipment. Pay attention to protect it from water.

Caution:

Keep the instrument clean and avoid oscillation.

Caution:

Please do not disinfect with high temperature or sterilize with electron beam and y radiation.

Caution:

Keep away from electromagnetic interference—Make sure that the operational environment is not interfered by relatively strong electromagnetic interference source, for instance X machine, multifunctional microwave therapeutic instrument and other equipments.

2

Caution:

Before use, user must check that the equipment has no obvious damages that may affect patient safety or instrument performance. The recommended inspection cycle is once a month or shorter time. In case of obvious damages, it is recommended to use the damaged component replaced previously.

Caution:

The following safety inspection must be executed by properly trained persons with certain knowledge and practical experience. In general, test shall be conducted once every two years or designated by public institution in accordance with the inspection procedures.

① Check that whether the equipment has mechanical and functional damages.

② Check that whether label related to safety is easily recognizable.

③ Verify whether equipment functions are consistent with those described on manual.

Caution:

After effective of service life, this instrument shall be disposed in accordance with local laws.

Caution:

After using, battery shall be disposed in accordance with local laws.

Caution:

This instrument is a handheld tool for the examination of fetal heart rate and it cannot replace standard fetal monitoring.

Caution:

This instrument can be used only after closing battery holder.

Caution:

In case of using battery, please do avoid short circuit or reversely install positive and negative electrodes.

Caution:

At the time of storage, please do not put metal object together with battery so as to avoid accidental short circuit.

Version number:V1.0

The latest revision date of this specification: 10-2016

3

CHAPTER 2 PROFILE OF FETAL DOPPLER

2.1 Overview

Fetal Doppler applies ultrasonic doppler principle to obtain the movement information of fetal heart from abdomen of pregnant women, enlarge the signals, And output from the built-in speaker, through the calculation to get value of the fetal heart rate.

2.2 Intended use

This instrument is applicable to listen to the fetal heart sounds from the bodies of pregnant women and obtain the value of fetal heart rate so as to provide reference for clinical diagnosis.

2.3 Product classification

This instrument is applicable to listen to the fetal heart sounds from the bodies of pregnant women and obtain the value of fetal heart rate so as to provide reference for clinical diagnosis.

A. Classification/Rule of classification

It belongs to IIa,Rule 10.

B. Classification in accordance with

protection against electric shock
It belongs to internal battery, BF type.

C. Classification in accordance with

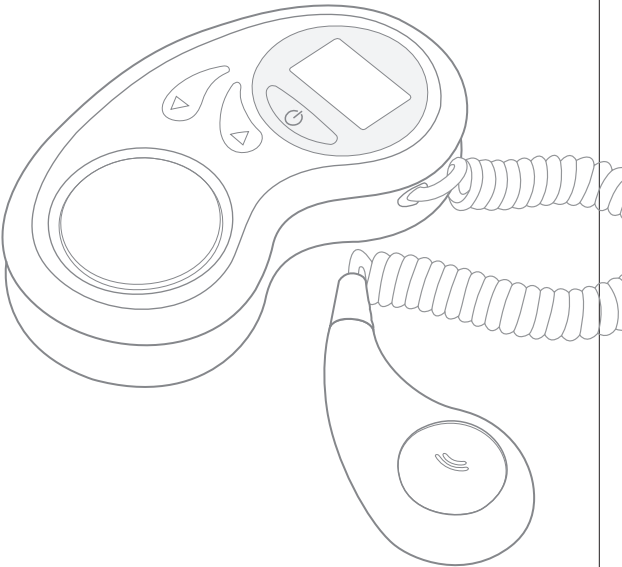
protection degree of harmful liquid inlet

D. Classification in accordance with
the safety degree used under the
situations with gas mixture of
flammable anesthetizing gas and air
or gas mixture of oxygen and nitrous
oxide.

The liquid inlet level shall be IP22.

It cannot be used under the situations
with gas mixture of flammable
anesthetizing gas and air or gas
mixture of oxygen and nitrous oxide.

4



Fetal Doppler

Operation Manual

2.4 Contraindication

Not yet found

2.5 Product structure

2.5.1 Product composition

The instrument is consisted of the host and the ultrasonic probe.

2.5.2 Product size and weight

Size	127(L)×65(W)×27(H)mm
Weight	Net weight is about 204g (including battery).

2.6 Performance requirements

2.6.1 Working environment

- Temperature: +5 C--+40 C
- Humidity: 30%-80%
- Atmospheric pressure: 60kPa-110kPa

2.6.2 Transportation and storage environment

- The packed instrument shall be stored at —20 C - +55 C.
- Relative humidity shall be 10%-93% (without condensation).
- In the room with atmospheric pressure of 50kPa - 100kPa, with no corrosive gas and good ventilation

2.6.3 Main parameters

2.6.3.1 Ultrasonic working frequency

Nominal sound working frequency: 2.0MHz/2.5MHz. (Specific to the probe tag)
The deviation between sound working frequency and nominal sound working frequency shall be less than ±5%.

5

2.6.3.1 Ultrasonic working frequency

Measuring range: 50-240bpm

Accuracy: ± 1% or ± 1 times / min,take a large value.

2.6.3.3 Integrated sensitivity

The sensitivity at the place 200mm away from the surface of probe: >90dB

2.6.3.4 Acoustic pressure at space peak value and time peak value P (MPa)

Negative acoustic pressure: P_<1MPa

Acoustic pressure at space peak value and time peak value: < 0.2MPa

2.6.3.5 Acoustic output power

Ultrasonic output power: < 40mW

2.6.3.6 Effective area of sensitive element of ultrasonic transducer

Effective area of sensitive element of ultrasonic transducer: 157mm²±10%

2.6.3.8 Operating mode: continuous wave

2.6.3.9 The service life is 3 years

2.7 Product Models

The products have five colors, namely pink, purple, blue,rose-bengal, and green.

6

2.8 Logos and signs



BF type applied part



Refer to operation manual



Manufacturer



Notice, please refer to attached documents



IP22—Level of protecting against liquid inlet

2.9 Manufacturer's Declaration of the EUT

Guidance and manufacturer's declaration – electromagnetic emission – for all EQUIPMENT AND SYSTEMS

1	Guidance and manufacturer's declaration – electromagnetic emission		
2	The Fetal Doppler is intended for use in the electromagnetic environment specified below. The customer or the user of Fetal Doppler should assure that it is used in such an environment.		
3	Emissions test	Compliance	Electromagnetic environment - guidance
4	RF emissions CISPR 11	Group 1	The Fetal Doppler uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
5	RF emissions CISPR 11	Class B	The Fetal Doppler is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
6	Harmonic emissions IEC 61000-3-2	N/A	
7	Voltage fluctuations / flicker emissions IEC 61000-3-3	N/A	

7

Guidance and manufacturer's declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration – electromagnetic immunity			
The Fetal Doppler is intended for use in the electromagnetic environment specified below. The customer or the user of the Fetal Doppler should assure that it is used in such an environment.			
Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment -guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity shouldbe at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	N/A	Mains power quality should bethat of a typical commercialor hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	N/A	Mains power quality should bethat of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles < 5 % UT (>95 % dip in UT) for 5 sec	N/A	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Fetal Doppler requires continued operation during power mains it is recommended that the Fetal Doppler be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic typical location in a typical commercial or hospital environment.
NOTE UT is the a. c. mains voltage prior to application of the test level.			

8

Statement: Maintenance data, including circuit diagram, component list, legends and correction, are only provided to qualified repairmen and units trained by the manufacturer.

Manufacturer information

Manufacturer: Xuzhou Yongkang Electronic Science Technology Co., Ltd
Registered address: 4F, Technology Pioneer Park C8, No. 40, Jingshan Road, Economic and Technological Development Zone, Xuzhou

Production address: 4F, Technology Pioneer Park C8, No. 40, Jingshan Road, Economic and Technological Development Zone,Xuzhou

After sales service address: 4F, Technology Pioneer Park C8, No. 40, Jingshan Road, Economic and Technological Development Zone,Xuzhou

Product Registration Certificate No./Product Technical Specification No.: SXZZ20162230095

Production liscence No.: SSYJXSCX 20050105

Shelf life: 3 years

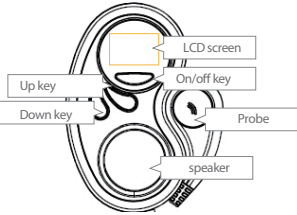
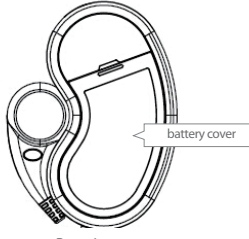
Manufacture Date: see the label

Tel: 0516- 87892766

Fax: 0516-82027233

Zip code: 221004

Websit: www.yonker.cn

Guidance and manufacturer’s declaration – electromagnetic immunity – for EQUIPMENT and SYSTEM that are not LIFE-SUPPORTING <table><tr><th colspan="4">Guidance and manufacturer’s declaration – electromagnetic immunity</th></tr><tr><td colspan="4">The Fetal Doppler is intended for use in the electromagnetic environment specified below. The customer or the user of the Fetal Doppler should assure that it is used in such an environment.</td></tr><tr><th>Immunity test</th><th>EN 60601 test level</th><th>Compliance level</th><th>Electromagnetic environment - guidance</th></tr><tr><td>Conducted RF IEC 61000-4-6</td><td>3 Vrms 150 kHz to 80 MHz</td><td>N/A</td><td rowspan="7">Portable and mobile RF communicationsequipment should be used no closer to any part of the Fetal Doppler including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \frac{3.5}{f^{1/2}} \sqrt{P}$$d = \left(\frac{3.5}{f} \right) \sqrt{P}$ 80 MHz to 800 MHz $d = \frac{7}{f^{1/2}} \sqrt{P}$ 800 MHz to 2.5 GHz where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).b Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,a should be less than the compliance level in each frequency range.b Interference may occur in the vicinity of equipment marked with the following symbol: </td></tr><tr><td>Radiated RF IEC 61000-4-3</td><td>3 V/m 80 MHz to 2.5 GHz</td><td>3 V/m</td></tr><tr><td colspan="3"></td></tr><tr><td colspan="3"></td></tr><tr><td colspan="3"></td></tr><tr><td colspan="3">NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.</td></tr><tr><td colspan="3">a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Fetal Doppler is used exceeds the applicable RF compliance level above, the Fetal Doppler should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Fetal Doppler. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.</td></tr></table> 9	Guidance and manufacturer’s declaration – electromagnetic immunity				The Fetal Doppler is intended for use in the electromagnetic environment specified below. The customer or the user of the Fetal Doppler should assure that it is used in such an environment.				Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment - guidance	Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	Portable and mobile RF communicationsequipment should be used no closer to any part of the Fetal Doppler including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \frac{3.5}{f^{1/2}} \sqrt{P}$ $d = \left(\frac{3.5}{f} \right) \sqrt{P}$ 80 MHz to 800 MHz $d = \frac{7}{f^{1/2}} \sqrt{P}$ 800 MHz to 2.5 GHz where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).b Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,a should be less than the compliance level in each frequency range.b Interference may occur in the vicinity of equipment marked with the following symbol: 	Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m										NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.			a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Fetal Doppler is used exceeds the applicable RF compliance level above, the Fetal Doppler should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Fetal Doppler. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.			Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM - for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING <table><tr><th colspan="4">Recommended separation distances between portable and mobile RF communications equipment and the Fetal Doppler</th></tr><tr><td colspan="4">The Fetal Doppler is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Fetal Doppler can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Fetal Doppler as recommended below, according to the maximum output power of the communications equipment</td></tr><tr><td></td><td colspan="3">Separation distance according to frequency of transmitter m</td></tr><tr><td>Rated maximum output of transmitter W</td><td>150 kHz to 80 MHz $d = \frac{3.5}{f^{1/2}} \sqrt{P}$</td><td>80 MHz to 800 MHz $d = \frac{1}{f^{1/2}} \sqrt{P}$</td><td>800 MHz to 2.5 GHz $d = \frac{7}{f^{1/2}} \sqrt{P}$</td></tr><tr><td>0.01</td><td>/</td><td>0.12</td><td>0.23</td></tr><tr><td>0.1</td><td>/</td><td>0.38</td><td>0.73</td></tr><tr><td>1</td><td>/</td><td>1.2</td><td>2.3</td></tr><tr><td>10</td><td>/</td><td>3.8</td><td>7.3</td></tr><tr><td>100</td><td>/</td><td>12</td><td>23</td></tr></table> For transmitters rated at a maximum output power not listed above the recommended separation distanced in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. CHAPTER 3 INSTALLATION AND OPERATION INSTRUCTIONS 3.1 Introduction of instrument 3.1.1 Appearance Please refer to Figure 1-1 for the detailed structures of all parts of instrument.  Left view  Main view 10	Recommended separation distances between portable and mobile RF communications equipment and the Fetal Doppler				The Fetal Doppler is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Fetal Doppler can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Fetal Doppler as recommended below, according to the maximum output power of the communications equipment					Separation distance according to frequency of transmitter m			Rated maximum output of transmitter W	150 kHz to 80 MHz $d = \frac{3.5}{f^{1/2}} \sqrt{P}$	80 MHz to 800 MHz $d = \frac{1}{f^{1/2}} \sqrt{P}$	800 MHz to 2.5 GHz $d = \frac{7}{f^{1/2}} \sqrt{P}$	0.01	/	0.12	0.23	0.1	/	0.38	0.73	1	/	1.2	2.3	10	/	3.8	7.3	100	/	12	23	 Right view  Rear view Figure 1-1 Front and rear view of instrument 3.1.2 Display The display of all parts of liquid crystal display is shown in Figure 1-2. The volume display Value of fetal heart rate  Battery level Unit of fetal heart rate Figure 1-2 Introduction of display content of liquid crystal display In the figure, 145 refers to the measured value of fetal heart rate, which means that fetal heart beats for 145 times. If there is no fetal heart rate, the corresponding value area displays horizontal line.  Battery level  The volume display  Unit of fetal heart rate Notice: Since liquid crystal display has limit visual angle range, please observe the display content of liquid crystal display from the front and keep the line of sight be perpendicular to liquid crystal display so as to ensure the best visual effect. In case of observing from other angle, the visual angle may cause blurring or reading error. 3.1.3 Keys  ON/OFF key  Up key :volume +  Down key :volume - 11	3.1.4 Battery 3 AA(No. 5) alkaline battery, 4.5V Service time of battery: not less than 60 minutes. 3.2 Product configuration list <table><tr><th>Article name</th><th>Quantity</th></tr><tr><td>Host</td><td>1</td></tr><tr><td>Manual</td><td>1</td></tr><tr><td>Certification</td><td>1</td></tr></table> 3.3 Instrument operation and usage 3.3.1 Turn on/off Before using fetal doppler to conduct examination, it is required to check whether there is any mechanical damage or crack on host, surface of probe and slingshot lines. In case of damage or normal sign, this instrument shall not be used for examination, please contact with manufacturer or dealer. Open the battery cover on the back of the instrument, install the battery in accordance with the direction in battery cabin, and close the battery cover. In the need of turning on, press the ON/OFF key of instrument “  ” and wait for the liquid crystal display to begin to display. At the very time, loosen the key and the instrument enters into normal working condition. In case of turning off, it only needs to press the ON/OFF key of instrument “  ” and wait for the liquid crystal display to turn off display. At the very time, loosen the key and the instrument is turned off. To prevent turning on/off caused by accidental operation, turning on/off of the instrument can be realized by continuously pressing the ON/OFF key for 1 second. Short time of pressing ON/OFF key will considered to be accidental touch and the instrument will not response. To prolong the service time of battery, if there is no value of fetal heart rate within continuous two minutes or no key operation, the instrument shuts down automatically. 3.3.2 Detection of fetal heart rate Before detection of fetal heart rate, the machine shall be opened firstly, the sound volume shall be adjusted to proper position, and appropriate amount of ultrasonic coupling agent is evenly printed on the acoustic area of probe. Ultrasonic probe is 12	Article name	Quantity	Host	1	Manual	1	Certification	1	aligned with the fetal heart position in the abdomen of pregnant women. Probe position and probe angle are adjusted properly until the most clear and resonant fetal heart sound can be heard. In general, the position of fetal heart with small gestational age is at the lower 1/3 of the pubis-umbilicus line. The position moves upward with the increase of gestational age and the position inclines left or right with the differences of fetal position. Since fetus moves in body, it may cause a wide range of movement for the position of fetal heart. Therefore, careful operation is needed to find accurate position of fetal heart. In case of moving probe, it is required to make sure that there is enough coupling agent between the surface of probe and the abdomen of pregnant women. If coupling agent is not enough, it may be impossible to hear fetal heart sounds or the heard fetal heart sounds are not clear, which affects the accuracy of calculation. In case of listening to fetal heart sounds, it is required to pay attention to the exclusion of blood flow sound in the abdomen of pregnant women. The frequency of this sound is obviously higher than that of fetal heart sounds. The value of fetal heart rate and the sounds of fetal heart measured are only used for clinical reference. If medical staff or users doubt it, other medical measures shall be taken for confirmation. When fetal doppler is used for pregnant women with too small gestational age or too fat pregnant women, the value of fetal heart rate may be inaccurate and the fetal heart sounds may be unclear. 3.3.3 Volume adjustment Adjust volume through pressing up key and down key, pressing up key add the volume , pressing down key reduce the volume. 3.3.4 The volume display Volnme of speaker, 8 steps can be adjusted. 3.3.5 Detection of battery level Fetal Doppler can monitor battery level automatically. When battery level displays as , battery shall be replaced timely. When battery level displays as , and begins to flicker, battery shall be replaced timely, or instrument may shut down automatically at any time due to the low battery level. 13
Guidance and manufacturer’s declaration – electromagnetic immunity																																																																																		
The Fetal Doppler is intended for use in the electromagnetic environment specified below. The customer or the user of the Fetal Doppler should assure that it is used in such an environment.																																																																																		
Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment - guidance																																																																															
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	N/A	Portable and mobile RF communicationsequipment should be used no closer to any part of the Fetal Doppler including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \frac{3.5}{f^{1/2}} \sqrt{P}$ $d = \left(\frac{3.5}{f} \right) \sqrt{P}$ 80 MHz to 800 MHz $d = \frac{7}{f^{1/2}} \sqrt{P}$ 800 MHz to 2.5 GHz where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).b Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,a should be less than the compliance level in each frequency range.b Interference may occur in the vicinity of equipment marked with the following symbol: 																																																																															
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m																																																																																
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies. NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.																																																																																		
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Fetal Doppler is used exceeds the applicable RF compliance level above, the Fetal Doppler should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Fetal Doppler. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.																																																																																		
Recommended separation distances between portable and mobile RF communications equipment and the Fetal Doppler																																																																																		
The Fetal Doppler is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Fetal Doppler can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Fetal Doppler as recommended below, according to the maximum output power of the communications equipment																																																																																		
	Separation distance according to frequency of transmitter m																																																																																	
Rated maximum output of transmitter W	150 kHz to 80 MHz $d = \frac{3.5}{f^{1/2}} \sqrt{P}$	80 MHz to 800 MHz $d = \frac{1}{f^{1/2}} \sqrt{P}$	800 MHz to 2.5 GHz $d = \frac{7}{f^{1/2}} \sqrt{P}$																																																																															
0.01	/	0.12	0.23																																																																															
0.1	/	0.38	0.73																																																																															
1	/	1.2	2.3																																																																															
10	/	3.8	7.3																																																																															
100	/	12	23																																																																															
Article name	Quantity																																																																																	
Host	1																																																																																	
Manual	1																																																																																	
Certification	1																																																																																	
3.3.6 Battery replacement In case of replacing battery, firstly take down the battery on the back of instrumentand take battery from the instrument. Put 3 No. 5 batteries into battery cabin and close the battery cover. In case of replacing battery, please pay attention to battery polarity, or it may cause the power-on failure of instrument. Warning: If it is not used within a period, the battery shall be taken out. Warning: The battery replaced or taken from the instrument shall be properly disposed in accordance with relevant national requirement. Please do not discard the battery in fire so as to avoid explosion. Matters after use: After it is used each time, please press “  ” key to shut it down, use soft cloth or paper towel to wipe clean the coupling agent on the surface of probe, and then the instrument is stored properly. Warning: As an equipment to examine fetal heart rate in a short time, Fetal Doppler is not suitable to conduct monitoring for fetus for a long time and cannot replace conventional fetal monitor. If user suspects the results measured by the instrument, other medical measures shall be taken for confirmation. Warning: This instrument cannot be used together with high frequency surgical equipment or fetal monitor. Two or more Fetal Doppler cannot be used at the same time. Warning: In normal use, this instrument and probe part cannot be used by immersing into water or other liquid. In case of performance evaluation, the end face of probe can be vertically immersed into water or other liquid by the depth of 3mm. Caution: If the skin at the place contacting with probe is broken or bleeding, it is forbidden to use the instrument. After being used for patient with skin disease, the probe shall be disinfected. 14	Caution: If the skin at the place contacting with probe is broken or bleeding, it is forbidden to use the instrument. After being used for patient with skin disease, the probe shall be disinfected. Caution: Since this instrument is easily to be affected by portable or mobile radio frequency communication equipments (for instance, cell phone) at runtime, it is required to avoid using portable or mobile radio frequency communication equipments near the instrument, or it may cause interference to the instrument so as to lead to the abnormalities of sound output, even abnormal measured value. Caution: This instrument is portable equipment. During the process of using, please operate carefully so as to prevent falling off and pay attention to the safety of instrument and personnel. Caution: When family users use this instrument, they shall read this operation manual carefully. If necessary, they shall consult doctors, distributors or manufacturer. Caution: The ultrasonic probe used in this instrument belongs to susceptible device. When used, please handle it with care. Please do not knock or impact. Pay attention to preventing accidental damages, including falling off. Notice: We suggest that the ultrasonic irradiation time for pregnant women shall be minimized under the premise of meeting clinical needs. Notice: This instrument cannot be used for the measurement of adult heart rate, or the accuracy of measurement results is not guaranteed. Notice: In case of using this instrument, it may generate a certain dose of electromagnetic radiation, which may disturb the electronic equipments or instruments near it. Notice: In case of any quality or technical problems of this instrument, we recommend against repairing it by user. In case users need technical data for the purpose of repair, including circuit diagram, component list and internal wiring diagram, they can contact distributors or manufacturer to obtain them. 15	Warning: It is not allowed to refit this instrument. Notice: Take account of the effects of dusts and lights on the instrument. Notice: Take account of the effects caused by children and pets. CHAPTER 4 MAINTENANCE 4.1 Maintenance Since the ultrasonic acoustic area of ultrasonic probe is very precise, it must be handled with care. After the instrument is used, the redundant coupling agent on ultrasonic probe must be wiped off. These maintenance measures can prolong the service life of instrument. Before use, user must inspect the equipment to check whether there is obvious damage that may affect patient safety or instrument performance. The recommended inspection cycle is one a month or a shorter time. The ultrasonic probe is immersed into conducting liquid to check whether there is crack on probe and damages between probe cable and its plug. In case of obvious damages, it is recommended to use the damaged component replaced previously. Periodic safety test shall be conducted for instrument to ensure the insulation of leakage current, including leakage current flow. The recommended test cycle is once every two years or test is conducted as designated by public institution in accordance with the inspection procedures. The accuracy of fetal heart rate is controlled by instrument and it cannot be adjusted by user. If the result of fetal heart rate is unbelievable, please use other methods, including using stethoscope, to verify it or contact local distributors or manufacturer for help. 4.2 Instrument maintenance This instrument needs no special maintenance in process of using. However, if the instrument is stored for over 1 month, the battery shall be taken out of the instrument, or the instrument is likely to be damaged due to battery leakage. Overall inspection shall be conducted for this instrument once every two years so as to ensure normal instrument functions and performance and make sure the safety and effectiveness of product. Inspection shall be conducted by manufacturer or unit with corresponding qualification. 16	4.2.1 Instrument cleaning Before cleaning the instrument, please shut down firstly and take out the battery. In case of cleaning the instrument, please use soft cloth to dip soapy water or clean water to slightly scrub the surface of instrument and probe. After the surface of instrument is dried up, it can be started up and used. Warning: Do not let any liquid enter into the instrument or gap, or immerse any part of the instrument into liquid. Notice: Please do not use strong solvent, including acetone. It is forbidden to use wear materials (for instance, steel wool) to clean the surface of instrument 4.2.2 Instrument disinfection Before disinfection, the instrument shall be fully cleaned. Disinfection method: 75% medicinal alcohol or other general disinfectant to scrub the surface of the host and probe of the instrument to ensure that all contaminated surfaces contact disinfectant. After the surface of instrument is dried up, it can be started up and used. This instrument cannot be disinfected with low temperature, high temperature or γ ray. CHAPTER 5 COMMON FAULTS AND TROUBLESHOOTING 5.1 Troubleshooting of common faults 5.1.1 Fault phenomenon: startup failure Please first make sure whether battery has sufficient electricity and the installation direction is correct. If it is unsure, please replace with new battery and restart it. If it is unable to be started after replacing with new battery, the instrument may be damaged, please contact manufacturer or distributors to handle it. 5.1.2 Fault phenomenon : no sound First, press up key to adjust the volume to the maximum position, insert and extract earphones again. Slightly knocking the surface of probe with hand, if earphones still have no sound, the instrument may be damaged, please contact manufacturer or distributors to handle it. 17	5.1.3 Fault phenomenon: Inaccurate value of fetal heart rate Since instrument displays the real-time fetal heart rate, there may be some difference between it and the average fetal heart rate obtained in accordance with conventional method. This phenomenon belongs to normal situation. The inaccurate value of fetal heart rate may be related with the inaccurate position of fetal heart. Other interference sources or interference signals (for instance, cell phone) near the instrument may also cause inaccurate value of fetal heart rate. The strong interference signals generated from user’s moving probe can also cause inaccurate value of fetal heart rate in a short time. If the value of fetal heart rate is still considered to be inaccurate through above judgment, please contact manufacturer or distributors to handle it. 5.1.4 Fault phenomenon: poor signal and bad sound First, make sure whether appropriate amount of coupling agent is used. Then, move the probe to find the position of fetal heart, make sure that there are no other interference sources or interference signals. Too small gestational age or too fat pregnant women may also have the situations of poor signal or bad sound. If the value of fetal heart rate is still considered to be inaccurate through above judgment, please contact manufacturer or distributors to handle it. 5.1.5 Fault phenomenon: short service time of battery Please use alkaline battery manufactured by legal manufacturer. Since carbon battery has a low battery capacity, its service time is short, which belongs to normal phenomenon. CHAPTER 6 WARRANTY AND MANUFACTURER INFORMATION 6.1 Warranty This instrument cannot be repaired by user and all repairs shall be conducted by technicians recognized by the company. The warranty period is one year (calculated from the purchasing date). The scope of warranty includes all equipment faults caused by the failure of material component or production process. Under warranty, all components with faults can be repaired and replaced for free. Artificial damage is not covered in warranty. 18																																																																														